

## Using the Multiwise Adapter

Every Wisepill pillbox is shipped with a lithium battery. The batteries can be charged inside the pillbox by simply using a USB charger and the cable provided. In some cases however, it is more convenient to charge the battery outside of the pillbox. The Multiwise Adapter allows one to charge 4 batteries simultaneously, outside of the pillboxes. The Multiwise Adapter can also measure the battery voltage.

### The kit contains the following:


- Multiwise Adapter
- 2 x connector boards (one spare)
- USB Charger rated at 2.1 Amps (10 Watts)
- USB cable

### Using the Multiwise Adapter to measure a battery's voltage

- The middle connector on the connector board is used for measurement.
- To measure a battery, the USB charger is not required.
- Connect the connector board to the Multiwise adapter.
- Plug the battery into the center connector of the connector board.
- The voltage reading will be displayed on the voltmeter.

### Using the Multiwise Adapter to charge a battery

- The outer 4 connectors on the connector board are used for charging.
- Connect the Connector board to the Multiwise Adapter.
- Connect the USB Charger to a suitable main supply.
- Connect the USB cable from the Charger to the Multiwise Adapter.
- All 4 outer connector LEDs will glow green if no battery is connected.
- Plug in the batteries that you wish to charge into one on the 4 outer connectors on the connector board.
- The LED opposite the battery will be red while the battery is charging and will turn green when fully charged.

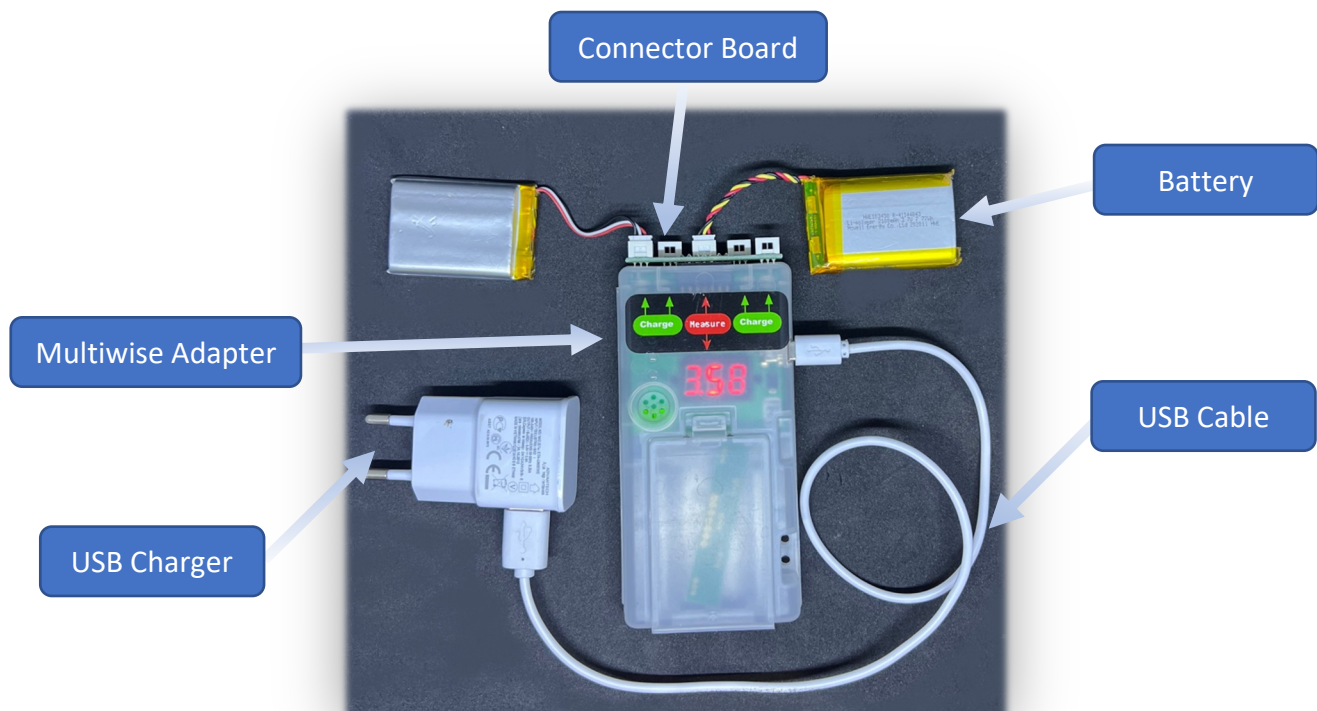

Using the Multiwise Adapter